



PARI BOY® Go

Quick Guide



Inhalation therapy with the PARI BOY® Go



1
Wash your hands thoroughly before each inhalation therapy session.



2
Fit the nebuliser top horizontally onto the controller.



3
Open the battery compartment and insert the batteries.



4
Alternatively, you can connect PARI BOY Go to the mains with a USB-C cable.



5
Open the nebuliser lid, add the inhalation solution, then close the lid.



6
Fit the mouthpiece or mask onto the nebuliser.



7
Sit up straight in a relaxed position. Switch on the device. Hold the mouthpiece between your teeth and close your lips around it.



8
If using a mask: the mask must fit snugly on your face. With the mask, you can also perform the inhalation therapy lying down.



9
Inhale slowly and deeply, and relax as you exhale. The device switches off automatically when all of the inhalation solution has been nebulised.

Cleaning and disinfection



Cleaning

Clean before first and after each use:

- Rinse the nebuliser top (with open lid), mouthpiece and mask under running tap water.
- Clean in warm tap water with washing-up liquid (about 5 minutes).
- Thoroughly rinse the parts under running tap water.



Cleaning

Wipe the outside of the controller and case with a clean, moist cloth.



Disinfection

Once a week after cleaning:

- In boiling, distilled water (at least 5 minutes).

Or

- In a standard thermal disinfectant (e.g. for baby bottles).



Drying

Place the individual parts on a dry, clean and absorbent surface after cleaning and/or disinfection.



Storage

Store the parts dry and dust-free in the case.

PARI care mode with automatic reminder function



Rinsing and drying the nebuliser top

Rinse the nebuliser top under running tap water, shake to remove excess water and check that the contact surface is dry. If necessary, dry the contact surface with a lint-free cloth and fit the nebuliser top onto the controller.



Fill with inhalation solution

Place the device with the rear side facing down and fill 15 to 20 drops of a 0.9 % saline solution or distilled water into the opening. Hold down the on/off button until you hear 2 beeps.



Automatic switch-off

The device switches off automatically after PARI care mode.

Please always observe the instructions for use when using and cleaning your PARI inhalation device.

Replacement

We recommend replacing the nebuliser top and mouthpiece once a year with a PARI BOY Go Year Pack.



For a video on easy handling, cleaning and care, visit www.youtube.com

PARI Service-Center

Simply give us a call or send an e-mail to export@pari.de for a prompt response. We are here to help!